

**Thank you for choosing  
Transcend 365 miniCPAP!**

We want to make sure  
you have the best experience  
possible as you begin your  
CPAP therapy. For additional  
information visit  
**[www.MyTranscend.com/  
support](http://www.MyTranscend.com/support)**

# TRANSCEND® 365 miniCPAP™



## **Components of the Transcend 365 miniCPAP™**

1. International cord kit
2. Transcend Travel Bag
3. Transcend 365 miniCPAP Quick Guide
4. Standard 6 foot hose (packaged separately)
5. Universal power supply (PSA3)
6. USB Cable
7. Transcend 365 miniCPAP with integrated heated humidifier



The Transcend 365 User Manual  
can be downloaded at  
**[www.MyTranscend.com/  
support](http://www.MyTranscend.com/support)**

## Assembling the Transcend 365 miniCPAP™

1. Connect your mask (sold separately) to an air supply tube with a standard 22mm connector.
2. Connect the air supply tube directly to the Transcend 365 miniCPAP device, making sure it is fully seated.
3. Connect the water reservoir to the device. Ensure that the silicone seal is not folded over or knocked loose while connecting.
4. Plug the power supply into the Transcend 365 miniCPAP and place on a flat, stable surface.
5. Connect the power supply to a wall outlet.

## Automatic Ramp

The ramp feature is enabled on your CPAP, unless your prescription states otherwise.

## Mask Settings

Be sure to set your device with the appropriate mask settings before you use it for the first time. This helps reduce the risk of mask leakage and ensures the correct amount of humidity is delivered during therapy. See the **Transcend 365 miniCPAP Quick Guide** for instructions.

## 365 User Tips:

- Never move the device with water in the reservoir. First remove the device from the reservoir then transport the device and reservoir separate.
- Place reservoir down on a flat surface and then place the 365 device on top, ensuring the silicone seal is not folded over or knocked loose.
- If storing for an extended period of time, allow device to fully dry before packing. This can take up to 24 hours.
- If the reservoir is not fully dry prior to transport, place in a Ziplock bag to prevent any water from damaging the CPAP.