



Batteries

User Manual

Table of Contents

Transcend Batteries	3
Preparing the Transcend Battery for First Use.....	3
Using the Transcend Battery as a Primary Power Supply	3
Using the Transcend Battery as Backup Power with AC Adapter.....	4
Transcend Batteries and LED Indicators	4
Transcend PowerAway P8 Battery and LED Indicators	4
Transcend P10 Battery and LED Indicators	5
Caring for Your Transcend Battery	5

Transcend Batteries

Transcend CPAP devices are only compatible with approved Transcend batteries.

Compatibility	Transcend P10 Battery	Transcend PowerAway P8 Battery
Transcend T365	✓	
Transcend T3		✓
Transcend Micro		✓

Preparing the Transcend Battery for First Use

The Transcend batteries are an optional power source for Transcend devices. Before first use, batteries need to be fully charged.

1. Remove the battery from its packaging.
2. Plug the end of the AC Power Supply cord into the battery.
3. Plug the AC Power Supply into an AC outlet. The battery will begin to charge automatically. It may take up to eight (8) hours to charge the battery.

Once the battery is fully charged, it is ready for use with your Transcend device. A fully charged battery charge is indicated when all the LED lights on the battery turn green.

NOTE: Do not connect the battery to your Transcend CPAP device during the initial charge.

Using the Transcend Battery as a Primary Power Supply

1. Insert the barrel connector of the battery into the power jack of your Transcend CPAP device.
2. Be sure the power-up LED flash sequence completes indicating that power is being supplied to your device, and that your device has successfully entered Standby Mode.

Using the Transcend Battery as Backup Power with AC Adapter

Using the Transcend battery in conjunction with AC power supply allows the battery to charge during therapy and provides backup power for uninterrupted therapy in the case of a power outage.

1. Insert the barrel connector of the battery into the power jack of your Transcend device.
2. Insert the barrel connector from the AC adapter into the battery.
3. Connect the AC adapter power cord into a power outlet.

Transcend Batteries and LED Indicators

Transcend PowerAway P8 Battery and LED Indicators

The Transcend PowerAway P8 Battery is an optional power source for the Transcend T3 and Transcend Micro.

- To use, connect the battery outlet cable to your compatible Transcend CPAP device by inserting the barrel connector into the power jack on the device.
- The LED on the battery indicates the charged state.

Red LED	A steady red light signifies a fault with the battery. Do not use or charge the battery when the red LED light is showing. Contact the CPAP device supplier who provided the battery to you for a replacement. NOTE: The battery will show a red LED light for two seconds when it is first plugged in.
Yellow LED	Indicates the battery is charging.
Green LED	Indicates the battery is fully charged. NOTE: The battery will show a yellow or green LED light when it is used in-line with the Power Supply

NOTE: The battery will show no LED light when it is used as the sole power source to power the Transcend Micro.

Transcend P10 Battery and LED Indicators

The Transcend P10 Battery is an optional power source for the Transcend 365.

- To use, connect the battery outlet cable to the Transcend 365 by inserting the barrel connector into the power jack on the device. The LCD will show the home screen when power is supplied to the device.
- When charging the P10 battery, the far-right LED indicates charging state.
 - When charging, LED is Green and flashing
 - When fully charged, LED is solid Green
 - When not charged or there is a fault, LED is Red.
- To check battery status, push and hold far left button for at least one second.
- The fuel gauge of five (5) LED lights indicates remaining battery power.

State of Charge	LED 1	LED 2	LED 3	LED 4	LED 5
Off	Off	Off	Off	Off	Off
1%-20%	On	Off	Off	Off	Off
21%-40%	On	On	Off	Off	Off
41%-60%	On	On	On	Off	Off
61%-80%	On	On	On	On	Off
81%-99%	On	On	On	On	On

Caring for Your Transcend Battery


- DO NOT immerse battery in water or cleaning solution.
- Wipe the surface with a dry, lint-free cotton cloth.
- For long term storage, which is any duration of time greater than three (3) months, the battery should be:
 - Stored at a temperature above -20C/-4F but below 20C/68F.
 - Stored in a dry location with a relative humidity below 70%.
 - Stored with a 30-40% charge.
 - Transcend P8: the charging time to get to 30% when the battery is depleted is 2 hours.
 - Transcend P10: battery will be at approximately 40% before the third LED turns solid green.

Notices



Notice	The information contained in this document is subject to change without notice.
Trademark	The moon logo, Transcend Micro, PowerAway, Somnetics, Transcend Inc., and Transcend are trademarks and/or registered trademarks of Transcend Inc.
Copyright	© Copyright 2022 Transcend Inc.; all rights reserved.

Contact Transcend Inc.

Corporate headquarters	Transcend Inc.
Email	customersupport@mytranscend.com
Web	http://www.MyTranscend.com/
Telephone	651.621.1800
Toll-free telephone	877.621.9626
Fax	651.204.0064
	 Transcend Inc. 103 Osborne Road NE Fridley, Minnesota 55432 USA

104206 Rev A 2022-05

NOTE: For more detailed descriptions, warnings, cautions and troubleshooting procedures refer to the Transcend CPAP device user manuals that are available for download at www.MyTranscend.com/support or by calling Transcend customer service at 877-621-9626 to request a copy.