

Drivers Install Instructions

Prior to installing the driver make sure that the CPAP is unplugged from the computer

1. You will only need to do these steps, if the boxes do not become colored when plugging in your CPAP
2. Go to <https://www.silabs.com/developers/usb-to-uart-bridge-vcp-drivers>
3. On this page you'll select downloads

OVERVIEW DOWNLOADS TECH DOCS COMMUNITY & SUPPORT

VCP Drivers Features and Benefits

The CP210x USB to UART Bridge Virtual COM Port (VCP) drivers are required for device operation as a Virtual COM Port to facilitate host communication with CP210x products. These devices can also interface to a host using the direct access driver.

These drivers are static examples detailed in Application Note 197: [The Serial Communications Guide for the CP210x](#).

The CP210x Manufacturing DLL and Runtime DLL have been updated and must be used with v 6.0 and later of the CP210x Windows VCP Driver. Application Note Software downloads affected are AN144SW.zip, AN205SW.zip and AN223SW.zip. If you are using a 5.x driver and need support you can download Legacy OS Software.

4. Download the CP210x Universal Windows Driver by clicking on it


Software • 10

CP210x Universal Windows Driver


v10.1.10

1/13/2021


5. Go to your downloads folder and double click on the file

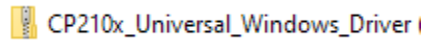


6. Click Extract All

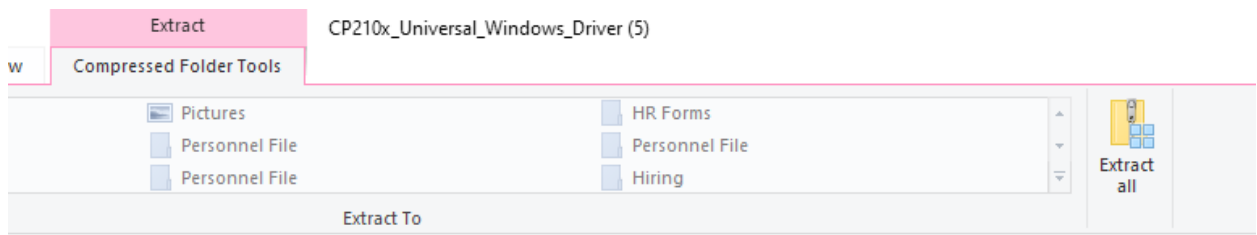


7. Leave the default folder and click extract

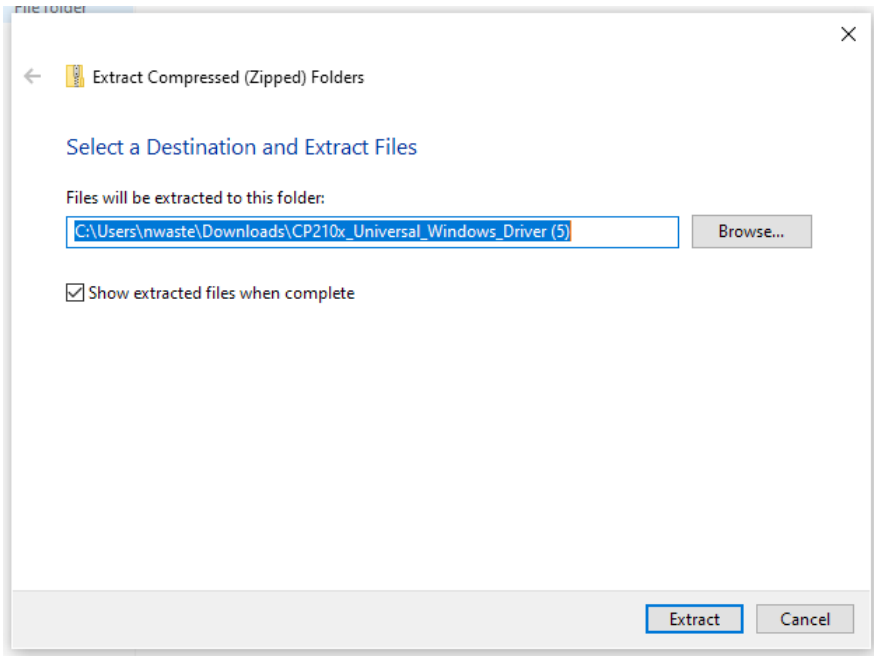

8. Find the file called silabser, there are 2 of them. One will have an icon of a piece of paper and gear. Right click on it and choose Install from the menu.

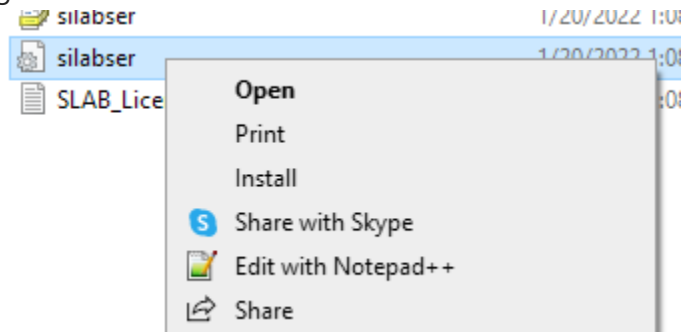

9. You will get a pop up that the operation completed successfully.

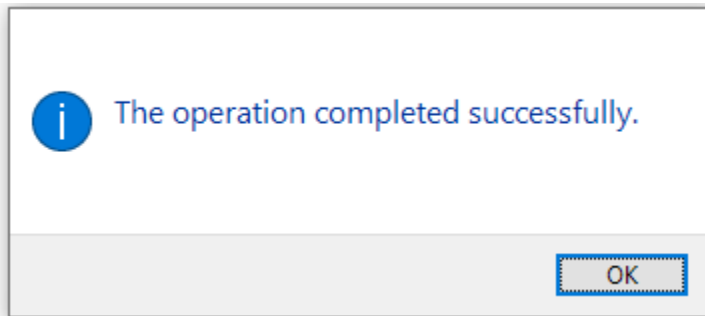

10. Reset the computer, open the MiniCPAP software and plug in the device