


Drivers Install Instructions

*****Prior to installing the driver make sure that the CPAP is unplugged from the computer*****


1. You will only need to do these steps, if the boxes do not become colored when plugging in your CPAP
2. Click [HERE](#) to download the drivers
3. Go to your downloads folder and double click on the file

 CDM-v2.12.36.4.U-WHQL-Certified (1).zip


4. Click Extract All




You can also click the right mouse button in the empty white space underneath the folders to Extract All.

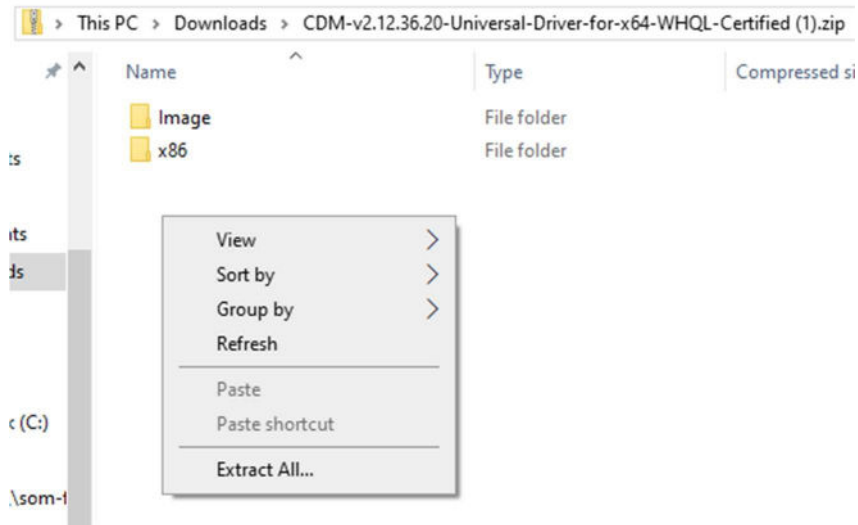

5. Leave the default folder and click extract

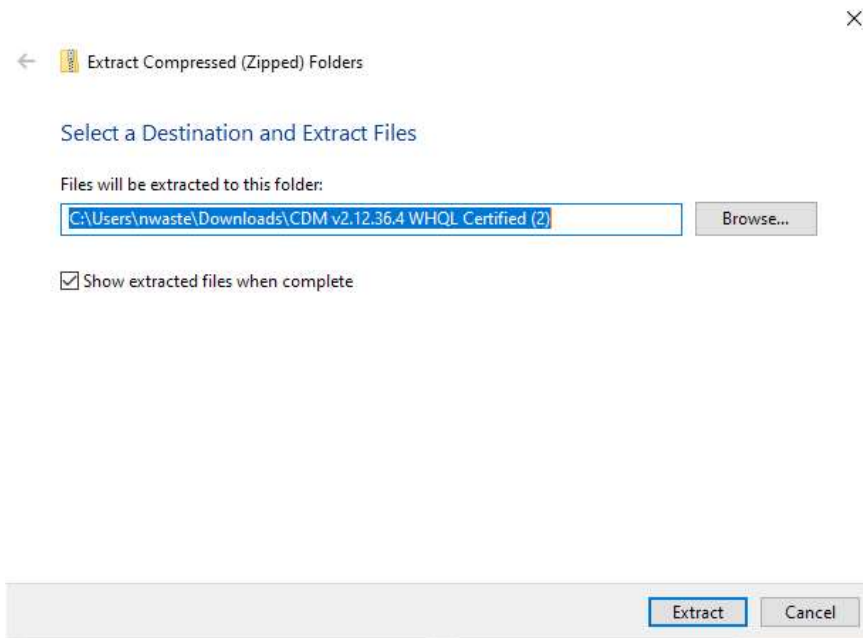

6. Open the Image folder.

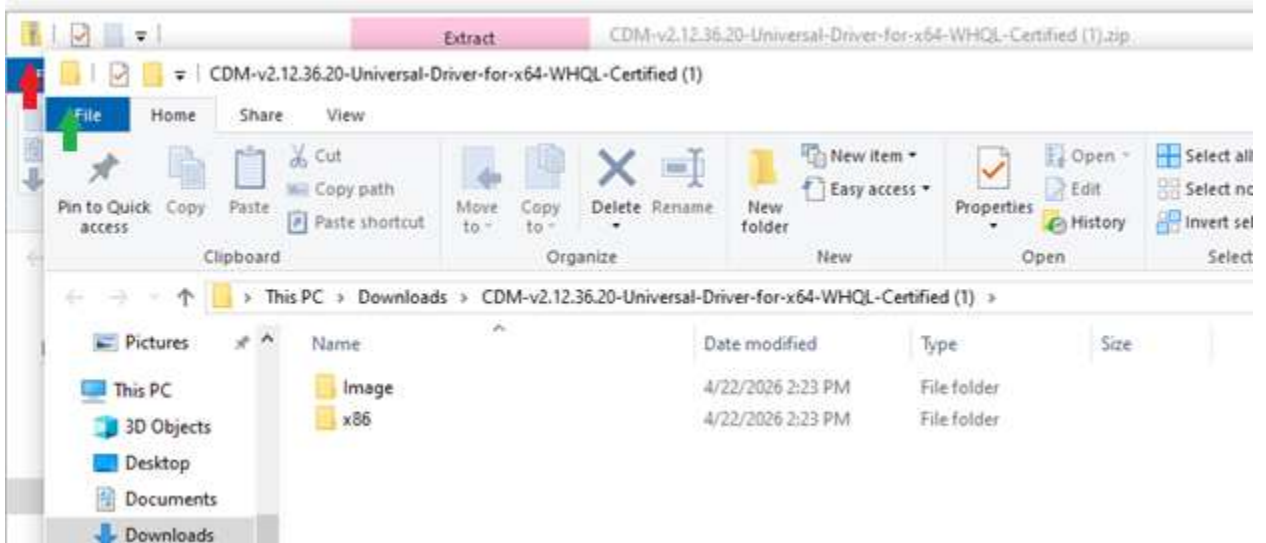


Also, please note that there are now 2 windows with similar names. If you look at the upper left corner, you can see a yellow folder symbol, one with a zipper and one without. Make sure you are on the one without the zipper or you will not be able to continue with the next steps.


7. Find the file called ftdibus, there are 2 of them. One will have an icon of a piece of paper and gear. Right click on it and choose Install from the menu.



8. You will get a pop up that the operation completed successfully.

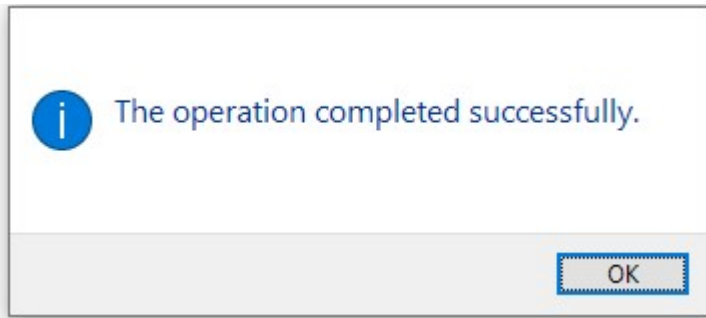

9. Find the file called ftdiport, there are 2 of them. One will have an icon of a piece of paper and gear. Right click on it and choose Install from the menu.

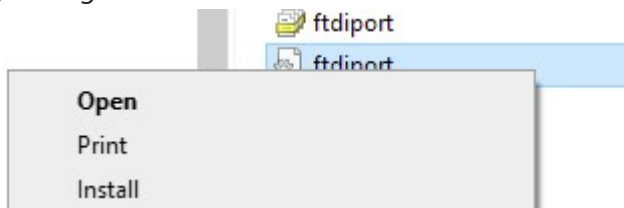

10. You will get a pop up that the operation completed successfully.

